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Sistema de implante dental Tree•Oss®

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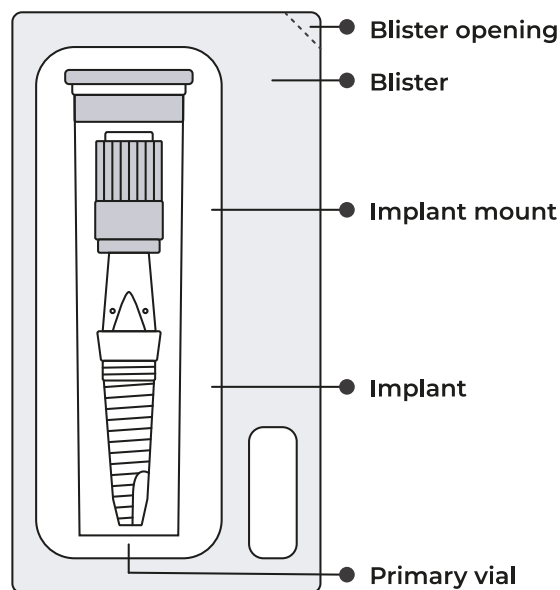
Visite www.tree-oss.com para mayor información.

 Terms and conditions of use disclaimer

Packaging Description

Dental implant system **Tree•Oss®**

Tree•Oss® dental implants are made of tetragonal Zirconia stabilized with Itria (Y-TZP), achieving a biocompatible product. They are sterilized by ethylene oxide.

The one-piece implant is made up of the implant itself and the ceramic abutment for the final restoration in the same piece.

Indication of Use

Designed to be implanted in the jaws and to support multiple or individual prosthetic structures.

Sterilization

Tree•Oss® dental implants are sold in sterile condition as indicated on the outer label. The sterilization is carried out by means of Ethylene

oxide (ETO) as indicated on the outer label. The sterilization is carried out by means of Ethylene Oxide (ETO).

Do not use the **Tree•Oss®** dental implants if the packaging is damaged or broken. Do not remove **Tree•Oss®** dental implants from their packaging until it is to be used. Under no circumstances should you re-sterilize.

Strict adherence to the custody chain of the product and an adequate asepsis by the professional in the clinical setting are essential for the product to be able to offer its full capabilities.

Storage conditions

Tree•Oss® dental implants have a 3 year shelf life, from the moment they are manufactured, when kept under normal storage conditions in cool places and with low humidity. The product should be stored in its original container, unopened and in a dry place at temperatures between 5°C and 50°C. Do not use dental implants after the expiry date.

Cautions and warnings

The implantation of **Tree•Oss®** dental implants should be performed by professionals who have completed specific courses in dental implantology in order to avoid the use of inappropriate surgical techniques and the incorrect evaluation of residual bone volume in

the maxillary, preventing early failure of bone integration as well as the loss of protein in bones. Under no circumstances should **Tree•Oss**[®] dental implants be used by professionals not trained for this purpose.

Should the professional not respect the limitations of use and indications, failures may occur, which means that the 100% implant success rate cannot be guaranteed. Implant treatment may result in bone loss or biological or mechanical failure, including fracture of implants due to fatigue. Close cooperation between the surgeon, the restorative dentist and the dental laboratory technician is essential for the success of the implant treatment.

The osteotomy in the area to be implanted should be performed following the sequence recommended in standard surgical protocols. Constant irrigation of the tooth during surgical procedures is highly recommended. We recommend that with the **Tree•Oss**[®] dental implants, you always use the instruments and prosthetic elements designed by **Tree•Oss**[®].

When removing the **Tree•Oss**[®] dental implant from its packaging, it should be noted that contact of the implant surface with elements other than zirconia is not convenient so as to avoid alteration and/or contamination of the implant surface. Likewise, it is recommended that the **Tree•Oss**[®] dental implant be placed as quickly as possible after removal from the container in order to prevent the adhesion of particles that are usually found in the air that may prevent proper bone integration and create infection risks.

Special attention should be paid to patients with local or systemic factors that may affect the healing process, either of bone or soft tissue, or the process of osseointegration (e. g. if the patient is a smoker, does not maintain adequate oral hygiene, has uncontrolled diabetes, is undergoing orofacial radiation therapy or steroid therapy, or has infections in the adjacent bone).

Usually, implant placement and prosthetic design should be tailored to individual patient conditions. In case of bruxism or unfavourable inter-maxillary relationships, a reassessment of the treatment option should be considered. A deficiency of hard or soft tissue before surgery may cause an unwanted aesthetic result or unfavourable implant angulations. Should the **Tree•Oss**[®] dental implant be placed using the technique indicated for two surgeries (submerged implant), before performing the last suture, ensure that the inside of the implant is free of blood residue. Due to the small size of the devices, care should be taken to ensure that the patient does not swallow or aspirate them.

The **Tree•Oss**[®] dental implants are not designed to be reused, as indicated on the outer label, because blood and bone residues left over from previous uses would lead to immediate rejection by the new patient's body.

Professionals should ensure that the patient is properly informed regarding the procedure and the post-surgical stages. The patient must be properly informed, in writing if possible, of the necessary care and the medication to be provided after the surgical procedure.

Never exceed an insertion torque of 45 Ncm during implant placement. Excessive tightening of the implant may cause damage to the implant, fracture or necrosis of the bone bed. If a surgical instrument is used to insert the implant, special care should be taken not to over-tighten the implant. The insert wrench used to torque the implant, when subjected to torque stress between the implant and the instrument, may cause marks on the implant. This is due to the different hardness of the materials. This does not compromise the integrity of the implant or biocompatibility, it is only aesthetic. If this happens, it is recommended to clean the implant pillar with alcohol for medicinal use.

A special feature of this single piece implant is the fact that the ceramic pillar for final restoration is already incorporated.

No part of the implant or abutment should be carved. The carving may cause the formation of micro cracks in the material, which could significantly reduce the implants resistance, causing a fracture of the implant during a normal occlusal loading. Additionally, there is also a risk of bone necrosis due to the excessive heat generated during the ceramic carving. Implants placed in the maxilla should not puncture the breast floor membrane.

Once implant placement surgery is completed, in order to ensure effective long-term treatment results, it is recommended that the patient be regularly followed up and informed of the appropriate oral hygiene procedures. In single-piece implants, the implant abutment should be protected against chewing and pressure against the cheek and tongue by means of a protective cover when primary stability is low. When good primary stability is achieved, a temporary piece can quickly be placed. For the final restoration, remove the temporary piece and clean the abutment by removing all cement remains. Do not use any abrasive pastes for cleaning. Use teflon-based cures and descaling products.

Prosthetic rehabilitation on **Tree•Oss®** implants should respect the basic biomechanical parameters as verified in this manual. Consider the appropriate number of implants placed according to the rehabilitated parts, the maximum lever arm, the occlusion and the correct alignment and distribution of forces. Avoid large lateral loads or bridges on implants, either between implants or on cantilever. In turn, a treatment plan with an oversized number of implants may also result in implant loss.

In the event that it is necessary to extract the crown, avoid damage to the shoulder or the abutment of the one-piece implant.

Zirconia dental implants are especially indicated in patients with allergies to metals and chronic diseases due to allergies to metals.

Contraindications

Planning and evaluation of the clinical case should be carried out prior to the surgical procedure to prevent patients from any risks associated with medical conditions that may develop into absolute or relative contraindications for surgery. In the first case, surgical

procedures should not be carried out, whilst in the second case, precautionary measures should be taken in order to avoid failure of the **Tree•Oss**® implants.

Tree•Oss® implants dental implants should not be placed in patients that are not in the necessary conditions to undergo general oral surgery. Do not use implants when the patient is being treated with bisphosphonate.

Tree•Oss® implants should not be placed or remain functional if the volume of healthy alveolar bone is not sufficient to support the implant minimally (minimum: 1 mm circumference around the whole implant and 2 mm apex) or if the minimum distance or distances between a one tooth and another implant are not respected (2 mm minimum distance with one natural tooth and 3 mm minimum distance from another implant).

Poor bone quality, poor oral hygiene by the patient, smoking, poorly controlled systemic diseases (diabetes, etc.), immunodeficiency, alcoholism, drug addiction and psychological instability may contribute to a lack of bone integration and the consequent failure of the implant, both immediately after surgery and after successful completion of the initial bone integration. Severe bruxism, prosthetic overload, loss of cervical bone, or excessive inclination of the implants that produce strong forces on the bone outside its axis, may result in bone loss, screw loosening, and rupture or implant failure.

Exposure to radiation and chemotherapy could affect the implants health. Patients with dental implants should be instructed to consult their doctor before receiving such treatments.

Possible contraindications for dental surgery are:

- ◆ Blood clotting disorders
- ◆ Systemic or local oral infections
- ◆ Diabetes
- ◆ Smoking
- ◆ Patients undergoing chemotherapy
- ◆ Alcoholism
- ◆ Kidney conditions
- ◆ Diseases that alter bone metabolism
- ◆ Untreated coronary or vascular conditions
- ◆ Insufficient residual bone
- ◆ Allergic or hypersensitive to zirconium dioxide (ZrO_2), yttrium oxide (Y_2O_3), hafnium dioxide (HfO_2) and aluminium oxide (Al_2O_3)
- ◆ Bruxism and related conditions

For further information on contraindications, please refer to clinical manuals and related text books.

Adverse reactions

Neither scientific publications nor other literature, have registered or informed on Zirconia rejection by the human body, or side effects resulting from the use of Zirconia. Failures occur during oral rehabilitation due to primary infections, contamination or incorrect sterilization of the instruments used during the surgical procedure, placement of the dental implant in areas with insufficient residual bone, or bone loss due to inadequate placement of the dental implant regarding the prosthetic rehabilitation performed.

The quality of **Tree•Oss**[®] implants, together with a correct observation by the intervening professional of the placement and handling techniques resulting from the above guidelines, guarantees the optimal functionality of the **Tree•Oss**[®] dental implants.

Description of the container and use of the product

The box includes an exclusive security label that guarantees the inviolability of the product until it is finally opened. The box also includes 3 (three) traceability labels (Implant card) and instructions for use.

The secondary packaging contains the primary packaging. To open this container it is recommended to follow the following steps:

- ◆ Remove the blister flap (check the 'Opening' area on the package image).
- ◆ Drop the tube into the sterile field (primary packaging).
- ◆ The primary packaging has been sterilized both inside and outside so it can be handled within the surgical field.
- ◆ Include sterilization witnesses.
- ◆ Include implant information: product code, lot and expiration date.
- ◆ Remove the implant from the tube by grasping it by the plastic handle.
- ◆ Bring the implant to the implantation site and thread a few coils.
- ◆ Remove the plastic handle by exerting a slight force against the implant.
- ◆ Placement is carried out with an accessory for manual use, until observing the need to proceed mechanically with a calibrated torque wrench.
- ◆ Continue with the osteotomy following the recommended surgical protocols.

Visit www.tree-oss.com web for more information.

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